

UnbornHeart™ Fetal Doppler App – Quick Guide to recording fetal heartbeat

Install the free app (iPhone, iPad, most Android phones and tablets). You can find the app at the Apple App Store and Google Play download links from www.unbornheart.com.



Install the batteries. Install the headphone pads. Connect the headphones to the Doppler and wear the headphones.



Connect the recording cable's 'DOPPLER' end to the Doppler. Connect the 'PHONE' end to your mobile device's audio jack.



Switch your mobile device to airplane mode. Start the app. Apply enough of ultrasound gel to your bare belly. Turn the Doppler on and you are ready to start monitoring your baby with the Doppler and recording the heartbeat with the app. Find more information in the app's built-in help.

After usage: Turn the Doppler off and wipe the ultrasound gel off from the Doppler with a soft tissue. Disconnect the recording cable from the Doppler and the mobile device. Switch your mobile device back to normal mode.

UnbornHeart™ app makes listening to your baby's heartbeat an entertaining and social experience by providing a way to record the fetal heart sounds and to share them with your family and friends. You can share the recorded heartbeat via email, text message, Facebook and Twitter. The app is not intended to be used for medical purposes but for entertainment purposes only.